

sensit
RAPID



Approved by

icmr
INDIAN COUNCIL OF
MEDICAL RESEARCH
Serving the nation since 1911



ubio
ubio ubiquitous.biology

SENSIT COVID-19

RAPID ANTIGEN TEST KIT

Accurate · Affordable · Accessible



"Test it right, in testing time"



Supported by,



जैवप्रौद्योगिकी विभाग
DEPARTMENT OF
BIOTECHNOLOGY



Marketed by,



Sprise India
Pvt. Ltd

SENSIT COVID-19

RAPID ANTIGEN TEST KIT

Accurate · Affordable · Accessible

Sensit COVID-19 Rapid Antigen Test Kit is a chromatographic immunoassay for the qualitative detection of specific antigens to COVID-19 present in human nasopharyngeal swab

FEATURES

- Works with nasopharyngeal swab from humans
- Visually read lateral flow assay
- Store/Transport temperature 4-30°C
- Assay time: 15 minutes
- Individually packed cassettes with desiccant

ASSAY PROCEDURE

- Add 15 to 20 drops of sample extraction buffer to the extraction tube
- Insert the patient swab sample into the extraction solution tube
- Then, swirl swab at least 10 times
- Remove the swab while squeezing the liquid from the swab by pressing tube side
- Cover the tube with a filter cap and tighten the lid
- Incubate the tube for 1-2 minutes
- Add 2 drops of the extraction solution with specimen into the sample well


Specifications

Method	: Immunochromatography	Assay time	: 10-15 min
Certification	: ICMR Approved	Shelf life	: 24 months
Sample type	: Nasopharyngeal swab	Storage temperature	: 4-30°C
Sensitivity	: 86%	Specificity	: 100%

INTERPRETATION

1. Positive
2. Negative
3. Invalid test
4. Invalid test

- The presence of any line, no matter how faint result is considered positive
- Positive results should be considered in conjunction with the clinical history and other data available

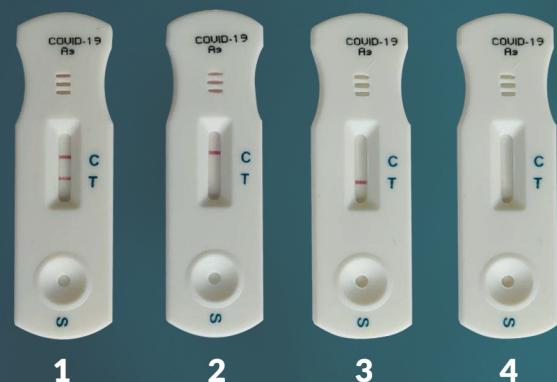

PRODUCT NAME	PACK SIZE
COVID-19 ANTIGEN RAPID TEST	10T, 25T, 50T

sensit
RAPID

Marketed by
Sprise India
Pvt. Ltd



ubio
ubiquitous.biology

ubio Biotechnology Systems Pvt. Ltd.
15A, KINFRA Hi-Tech Park
Kalamassery, Kochi - 683503 INDIA
www.ubio.in | contact@ubio.in
+91 97441 22269



* Disclaimer: For screening/ surveillance purposes only.
All tests must be confirmed as per ICMR guidelines.